

응용 자료

## ACQUITY UPLC I-Class/Xevo TQ-S micro IVD System: Analytical Performance for Phylloquinone

---

Stephen Balloch, Lisa J. Calton, Gareth Hammond

Waters Corporation

*For in vitro diagnostic use. Not available in all countries.*

This is an Application Brief and does not contain a detailed Experimental section.

---

### Abstract

Vitamin K1 (phylloquinone) analysis using electrospray ionization mass spectrometry is challenging due to the hydrophobic nature of the molecule and lack of ionization sites. An additional issue is the low concentration of vitamin K1, which may be as low or even lower than 0.1 ng/mL in serum.

A new clinical research method for the analysis of vitamin K1 in serum has been developed using UPLC-MS/MS with electrospray ionization. 200 µL sample was processed with ethanol and centrifuged. The supernatants were loaded onto Oasis PRiME HLB µElution plates, washed, eluted, and a solvent exchange carried out prior to analysis.

### Benefits

- A lower limit of the measuring interval (LLMI) of 0.05 ng/mL in serum
-

- Short run time (3.7 minutes injection-to-injection)

---

## Introduction

The Waters ACQUITY UPLC I-Class/Xevo TQ-S micro IVD System enables the quantification of organic compounds in human biological liquid matrices.

This document describes a test of the analytical performance of the ACQUITY UPLC I-Class/Xevo TQ-S micro IVD System for the analysis of vitamin K1 in serum.

---

## Results and Discussion

A chromatogram illustrating the chromatography of the vitamin K1 analysis is shown in Figure 1. Performance characteristics of vitamin K1 on the ACQUITY UPLC I-Class/Xevo TQ-S micro IVD System are shown in Table 1.

Compound	Range (ng/mL)	LLOQ (ng/mL)	%RSD at LLOQ	Total precision	Repeatability	EQA mean bias
Vitamin K1	0.1–20	0.05	7.6%	≤5.4%	≤4.4%	7.5%

---

*Table 1. Performance characteristics of vitamin K1. Range defined by linear fit where  $r^2 > 0.995$ . LLOQ defined by  $S/N (PtP) > 10$  and  $\%RSD \leq 20\%$ .  $\%RSD$  at LLOQ determined through analytical sensitivity experiments performed over five occasions ( $n=50$ ). Total precision and repeatability of QCs performed over five occasions in serum ( $n=25$ ). EQA mean bias determined by comparison of obtained values to All Laboratory Trimmed Mean.*

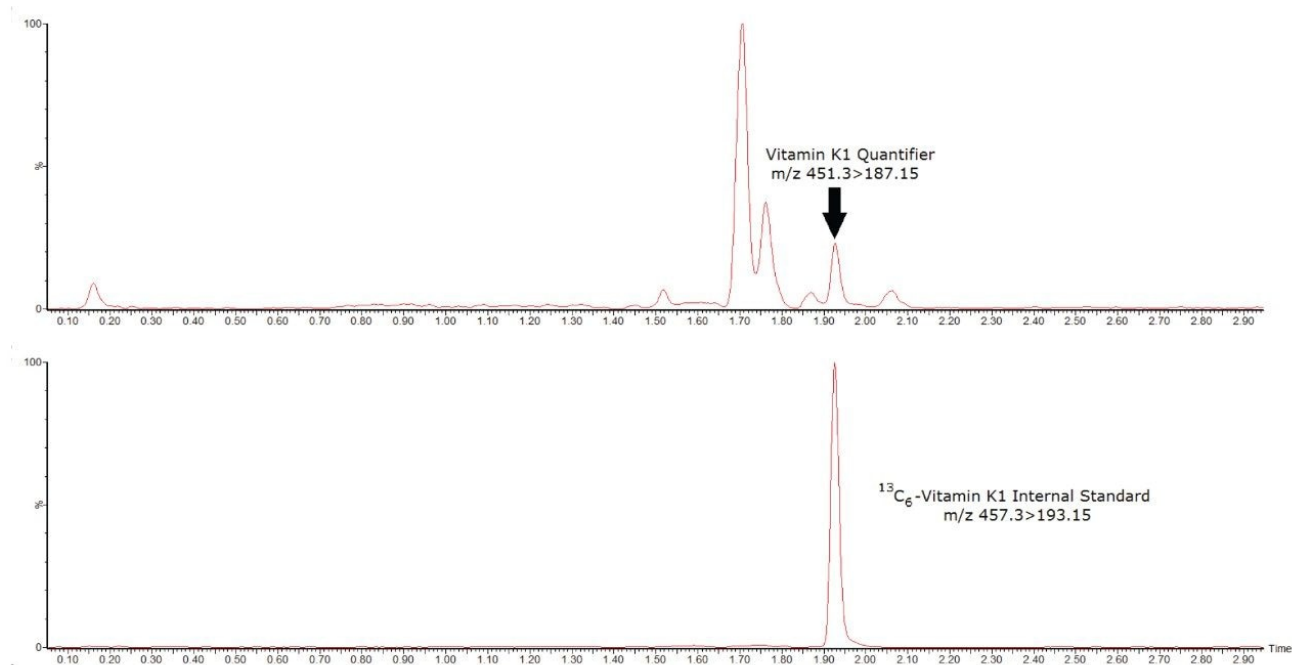


Figure 1. Chromatogram showing the analysis of a sample containing 0.14 ng/mL of vitamin K1 using the ACQUITY UPLC I-Class/Xevo TQ-S micro IVD System.

## Conclusion

The Waters ACQUITY UPLC I-Class/Xevo TQ-S micro IVD System has demonstrated the capability to deliver analytically sensitive, accurate, and precise performance for vitamin K1 in serum.

## Disclaimer

*The analytical performance data presented here is for illustrative purposes only. Waters does not recommend or suggest analysis of the analytes described herein. These data are intended solely to demonstrate the performance capabilities of the system for analytes representative of those commonly analyzed using liquid chromatography and tandem mass spectrometry. Performance in an individual laboratory may differ due to a number of factors, including laboratory methods, materials used, intra-operator technique, and system conditions. This document does not constitute a warranty of merchantability or fitness for any particular*

*purpose, express or implied, including for the testing of the analytes in this analysis.*

---

## Featured Products

[Xevo TQ-S micro Triple Quadrupole Mass Spectrometry <https://www.waters.com/134798856>](https://www.waters.com/134798856)

[ACQUITY UPLC I-Class PLUS System <https://www.waters.com/134613317>](https://www.waters.com/134613317)

[MassLynx MS Software <https://www.waters.com/513662>](https://www.waters.com/513662)

[TargetLynx <https://www.waters.com/513791>](https://www.waters.com/513791)

720007170, February 2021

© 2022 Waters Corporation. All Rights Reserved.

[Terms of Use](#)  
[설정](#)

[Privacy](#)

[Trademarks](#)

[Sitemap](#)

[Careers](#)

[쿠키](#)

[쿠키 기본](#)